

How to use

# 3M™ Curos Tips™ Disinfecting Cap for Male Luers

Disinfects in 1 minute. Protects male luers for up to 7 days.



Peel Curos Tips disinfecting cap from foil strip



Push and twist the Curos Tips disinfecting cap clockwise onto the end of the male luer

- If male luer connection on I.V. line is heavily soiled, clean with 70% isopropyl alcohol swab before using Curos Tips disinfecting cap
- Do not use if foil is broken, torn, punctured, or if Curos Tips disinfecting cap appears dry inside
- Curos Tips disinfecting cap must remain on male luer connection for a minimum of one (1) minute and may remain on for seven (7) days if not removed
- The needleless connector can be accessed immediately after removing the Curos Tips disinfecting cap that has been in place for a minimum of one (1) minute
- Always place a new Curos Tips disinfecting cap on male luer connection after each use
- Dispose of the Curos Tips disinfecting cap after every use; do not reuse

**Caution:** Potential choking hazard.



Green means clean

**STERILE | R**

Active Ingredient: 70% Isopropyl Alcohol (IPA). Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M is a trademark of 3M. All other trademarks are owned by their respective owners.

**Solventum Medical Surgical**

**Phone** 800-228-3957

**Web** [Go.Solventum.com/Curos](https://Go.Solventum.com/Curos)

70-2011-5798-2