


How to use

3M™ Curos™ for Tego®

Disinfects in 1 minute. Protects ports for up to 7 days.



Peel the foil from the Curos Disinfecting Cap for Tego® Hemodialysis Connector

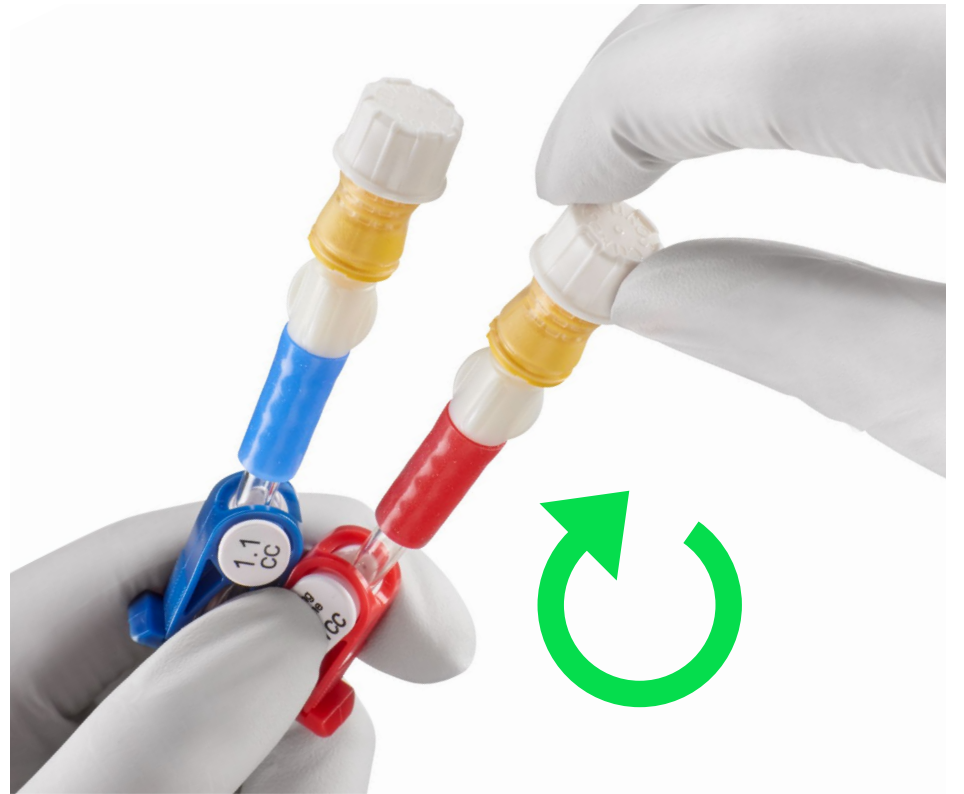

Push and twist the Curos disinfecting cap for Tego® hemodialysis connector clockwise onto the end of the hemodialysis connector

- If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab prior to using cap
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry
- Cap must remain on needleless connector for minimum of one (1) minute and may remain on for seven (7) days if not removed
- The needleless connector can be accessed immediately after removing the Curos disinfecting cap for Tego® hemodialysis connector that has been in place for a minimum of one (1) minute
- Always place a new Curos disinfecting cap for Tego® hemodialysis connector on hemodialysis connector after each use
- Dispose of the Curos disinfecting cap for Tego® hemodialysis connector after every use
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses

Warning: To avoid potential for injury, use only Tego® Hemodialysis Connectors.

Caution: Potential choking hazard.

STERILE | R

Active Ingredient: 70% Isopropyl Alcohol (IPA). Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.

© Solventum 2025. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M, the 3M logo and other 3M owned and/or licensed trademarks are trademarks of 3M. Other trademarks are the property of their respective owners.

70-2011-5799-0