

How to use: Disinfecting cap for stopcocks and catheter hubs

3M™ Curos™ Stopper Disinfecting Cap for Open Female Luers

Disinfects in 1 minute. Protects open female luers for up to 7 days.



Peel



Push and twist on stopcock



Push and twist on catheter hub

- Verify stopcock valve or catheter clamps are in the closed position prior to using the Curos Disinfecting Cap for Open Female Luers
- If open female luer is visibly soiled, clean with 70% Isopropyl Alcohol swab before using cap
- Do not use cap if the seal is broken, torn, or punctured
- Detach the cap from the foil
- Once foil is removed, push and twist device clockwise onto the end of the open female luer until tightly connected
- Cap must remain in place for a minimum of one (1) minute and may remain on for seven (7) days if not removed
- The open female luer can be accessed immediately after removing the Curos Stopper disinfecting cap that has been in place for a minimum of one (1) minute
- Discard cap after each use; do not reuse cap
- New cap should be placed on open female luer after each use
- Follow hospital protocol for disinfecting open female luers, if performing sequential accesses

Warning: To avoid potential for injury, use only on open female luers and ensure the cap is securely tightened.

Caution: Potential choking hazard.



STERILE R

Single Use Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.

© Solventum 2025. Solventum, the S logo and other trademarks are trademarks of Solventum or its affiliates. 3M, the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners.

70-2011-6729-6